



Resin		(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
Cast Crown	ProCast Denture	5	13	8	ON
Guide		6	10	8	ON
Tray		7	13	8	ON
Splint		10	6	8	ON
Ortho Model	Model ProDenture	10	13	8	ON
Temp	IBT				
ProSplint		15	8	1	ON
Element Model Beige	Element Model White	20	2	5	OFF
Gingiva		20	13	8	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
C&B	5	13	8	ON
Denture	10	13	8	ON
Hard Splint	5	8	8	ON



Resin		(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
Guide		5	16	10	ON
Tray		7	16	10	ON
FLEX	Model	10	16	10	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
Model Temp	10	13	8	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
VarseoSmile Crown plus VarseoSmile Teeth VarseoSmile Temp VarseoSmile TriniQ	2min 30s*	13	8	ON

*** Flip the object and restart the post curing cycle**

Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
3Delta Model 320 3Delta Model Grey	6	14	10	ON
3Delta Gingiva Mask	10	14	10	ON
3Delta Cast M	8	14	10	ON
3Delta Cast P	3	14	10	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
Crown & Bridge Denture base Denture teeth Try-in	10	13	8	ON

Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
Cast Lumina Zero Denture Cast NF	5	13	8	ON
Guide	6	10	8	ON
Tray	7	13	8	ON
Splint	10	6	8	ON
IBT Temp Match Model Grey Mock up Model GB Model Ivory Ortho Model Laviva light-pink	10	13	8	ON
Clara	15	8	1	ON
Gingiva	20	13	8	ON





Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
Tray	7	13	8	ON
Gingiva Model Beige Model Grey Model Transparent	10	13	8	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
Guide Model2 beige Model2 white	10	13	8	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
C&B Cast	5	13	8	ON
Model Model Basic	10	13	8	ON
Surgical Guide	6	10	8	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
Ocean Blue	5	13	8	ON
Beluga White Orca Black Dolphin Grey	10	13	8	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
KeyGuide Key Try In	5	16	10	ON
KeyMask KeyModel KeyModelUltra Ivory KeyModelUltra Grey KeyOrtho IBT KeyOrthoModel KeySplintSoft KeyTray	10	16	10	ON
KeySplint Hard	12	8	5	ON
KeySplint Hard Clear	10	8	5	ON
KeySplint Soft Clear	10	10	10	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
Cast Denture	5	13	8	ON
Guide	6	10	8	ON
Tray	7	13	8	ON
Splint	10	6	8	ON
Model GB Ortho Model	10	13	8	ON
Temp IBT				
Gingiva	20	13	8	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
CMFH Denture 3D+	5	13	8	ON
Tray	7	13	8	ON
SG Ortho IBT Model 2.0	10	13	8	ON
Ortho Flex	10	16	10	ON

NYTE 3D

Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
Cast Press Model Sand	10	13	8	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
Splint	10	16	10	ON
Sculpture Sculpture 2.0 SG Titan	20	9	2	ON
Bite Tray Denture Try in IBT Model	20	12	10	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
CROWNTEC	3*	16	10	ON

*** Flip the object and restart the post curing cycle**



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
Cast & Press	5	13	8	ON
Tray	7	13	8	ON
Gingiva Model Plus UV Desert Model UV Smoky white Model Plus UV Stone	10	13	8	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
P Pro Master Model Dark Beige P Pro Master Model Grey	6	14	10	ON
P Pro Gingiva	10	14	10	ON
P Pro Cast M	8	14	10	ON
P Pro Cast P	3	14	10	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
Cast	9	13	8	ON
Dentbase Model 2.0 SG Splint comfort	7	13	8	ON
Model Beige	5	16	10	ON
Model Fast Splint Tray Try-In	5	13	8	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
VeriModel Golden Brown VeriModel Grey VeriModel Ivory VeriModel White	5	16	10	ON
VeriTray	7	13	8	ON
VeriCast VeriSplint VeriGuide	10	16	10	ON
VeriGUM	2	16	10	ON



Resin	(T) Time(min)	(P) Intensity	(D) Duty Cycle	(B) BL
TrinDy Tray TrinDy Model	10	13	8	ON